


Able2

Forearm Crutch Series

PR30081 / 10081C, PR30082 / 10082C
10071C, 10072C, 10073C, 10074C, 10075C, 10076C

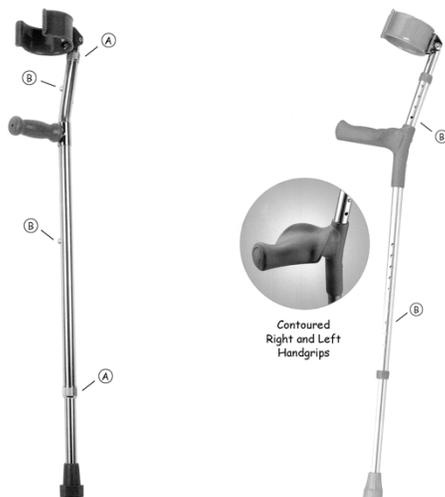

Please inspect all parts carefully for cracks/breaks prior to use.

ASSEMBLY/USER INSTRUCTIONS

Forearm Crutches come fully assembled.

Forearm Crutches are used for partial weight bearing to help you balance or redistribute the load placed on your legs while you walk.

These crutches have two adjustments, one for the distance between the cuff and the handgrip and one for the shaft length.

Before making any tubing length adjustments loosen the threaded tightening ring (A) (if there is one, some models do not have a threaded tightening ring) on the crutch shaft.

Distributed By:

Able2 Europe B.V.
Koperslagerij 15
4762 AR Zevenbergen
The Netherlands
www.able2.nl

Able2 UK Ltd.
www.able2.eu

Able2 Iberia S.L.
www.able2.es



When adjusting the length of the shaft tubing, be sure the push button (B) fully protrudes through the adjustment hole of the shaft to ensure a positive lock. Once you have the desired height tighten the threaded locking ring.



Do not hang bags on the handle of your crutch.

MAINTENANCE INSTRUCTIONS


Before use, check to make sure that rubber tips are not ripped, worn, or missing. Replace if necessary before use.

CARE INSTRUCTIONS

Clean with mild, non-abrasive cleanser and water, dry thoroughly.




Maximum
User Wt:

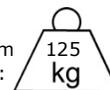

Series: 10071C, 10072C, 10073C, 10074C, 10075C, 10076C



Maximum
User Wt:

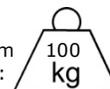

Series: PR30081/10081C, PR30082/10082C



Valentine International Ltd.
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C.



Y. Sung Handelsvertretung
Duesselthaler Str. 24
40211 Duesseldorf Germany



Made in China

Rev. 071520