

From walking stick to restful seat in an instant!

Thank you for choosing a Linden Trio seat. We wish you many years of happy use. For your complete safety please read all instructions:

- Trio Popular Walking height: 845mm (33¹/₂") Seat height: 455mm (18")
- Trio Maxi Walking height: 870mm (34¹/₄") Seat height: 575mm (22¹/₂")
- Maximum user weight: 125kg

Your Trio seat is complete with three rubber ferrules and has been designed for use on level, non slippery surfaces.

N.B. Regularly check that all rubber ferrules are in good condition. For your safety NEVER use the seat with any ferrule missing.

- 1 . To use as a seat: Open fully and ensure all feet are in contact with the ground.
- Sit down slowly. When sitting always have the handle behind you and both your feet on the ground. (see picture).
- The Trio seat is intended for use only as a one person seat. Do not use it for anything else.

SPORT SEAT MODELS

The Sport seat models with the additional point and plate feet can be used on soft ground by removing all rubber ferrules. Replace all ferrules for use on hard ground.



MODELS AVAILABLE

Polished aluminium frame

or

'Deluxe'

painted frame to match seat colour –



